

Measurement Chart for Medipaw Slim Boots





Getting the right size for your patient.

1. Place sizing chart on firm surface.
2. Place paw on sizing chart and align the edge of paw with **bold line labelled A**
3. Lift patient's opposite paw so that weight is placed firmly onto paw being measured.
4. Mark BOTH SIDES of weighted paw at its widest point on the sheet of paper
5. Measure the distance and choose correct boot size (if you are between sizes, choose the smaller size).

AVAILABLE SIZES						
5.08cm	6.35cm	6.35cm	7.62cm	8.255cm	9.525cm	11.43cm
X-Small	X-Small 2	Short	Small	Medium	Large	X-Large
(Yellow)	(Pink)	(Orange)	(Blue)	(Red)	(Green)	(Purple)

First, print the Sizing Chart at 100% size. After printing, verify the accuracy of this chart with a ruler before measuring.

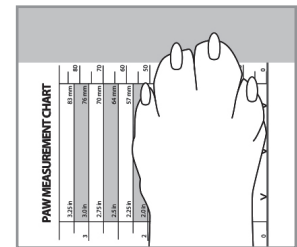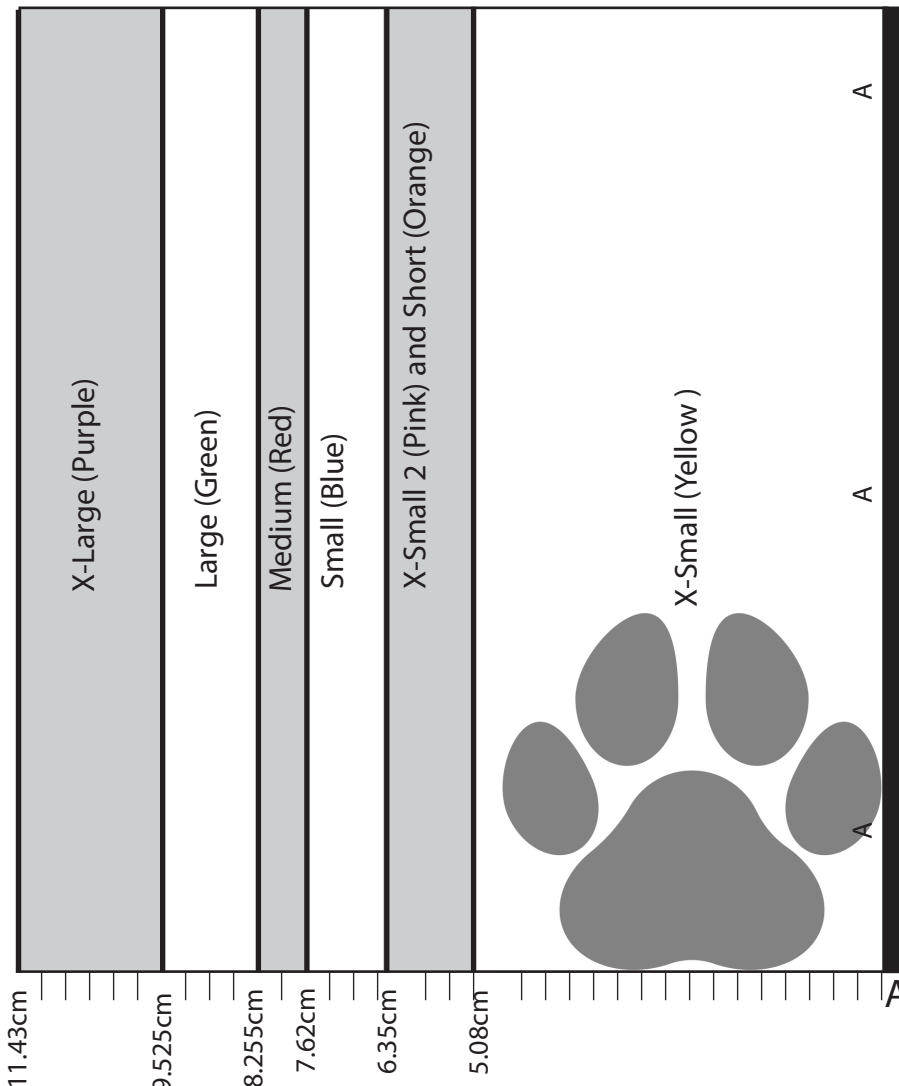

PLACE EDGE OF PAW ALONG THIS LINE

Questions?
 Call or email us at
info@medivetproducts.com
 P: 800.245.3413
 F: 646.783.3808

www.medivetproducts.com

Protective Boot
 U.S. Patent No. D747,048 S